

Product information: 20570

wiping disinfection

- ready to use and fast acting disinfection, alcohol-free
- soaked extra large disinfection wipe
- up to 3 m² are disinfected with one wipe
- improved safety due to remaining effect
- excellent bactericidal, fungicidal and viricidal activity
- aldehyde- and phenol-free
- skin-friendly - was awarded highest marks when dermatologically tested
- already effective after 1 minute



CE
0481

RHEOSEPT-WD plus Wipes

Alcohol-free disinfection wipes

Product description

Soaked disinfection wipes. Already effective after 1 minute. Very good cleaning ability. Gentle to the material. Suitable for waterproofed surfaces incl. acryl glass. Pleasant scent. Full dispensing of active soaking liquid in order to assure disinfection. No consumption of active liquid by the wipe-material.

Fields of application

Fast-acting disinfection by means of a wipe process for all water-resistant surfaces of medical products such as device surfaces, seats, examination tables, pans as well as sunbeds. Suitable for materials such as plastics, lacquers, ceramics, acrylic glass etc. Special suitability assessments for plexi glass and interior fitting of ambulance vehicles are available.

Dosing instructions

Wipe surfaces thoroughly. Take care that surfaces are completely moistened in order to assure complete disinfection. After residence time use a one way cloth. After use of wipes dispose them accordingly. Wear of gloves is recommended, Do not use for skin-cleaning!

Spectrum of activity

Spectrum of activity: bactericidal, fungicidal, viricidal (HIV, HBC, HCV, noro, rota, vaccinia, BVDV).

Contact times

Dosing specifications	Dosing details	Remark
contact time 60 seconds	undiluted	bactericide/fungicide EN 1276, EN 13697, EN 1650
contact time 5 minutes	undiluted	bactericide/fungicide (VAH) low load/mechanics
contact time 60 seconds	undiluted	enveloped viruses (HIV, HBV, HCV, BVDV, vaccinia, influenza) EN 14476
contact time 30 minutes	undiluted	noro-virus (MNV) EN14476
contact time 60 seconds	undiluted	rotavirus
contact time 5 minutes	undiluted	polyomavirus SV40

Environmental information

Wachendorff-Chemie GmbH produce under application of advanced, safe and environmentally friendly procedures and in compliance with high quality standards.

Composition

100g contain: 0,25g dialkyldimethylammoniumchloride, 0,25 g benzalkoniumchloride, 0,25 g alkyldimethyl(ethylbenzyl)ammoniumchloride. Further ingredients: scents, limonenes.

Physico-chemical data

- pH-value 7 (application)

Forms of delivery

REF	Package size	Packaging unit
20570003	RHEOSEPT-WD plus Wipes 48 pieces (300x270 mm)	carton
20570011	RHEOSEPT-WD plus Wipes mini 30 pieces (200x180 mm)	carton

Registrations

This medical product is CE certified (CE 0481).

Besondere Hinweise

This product is not a hazardous material in accordance to the Hazardous Substance Ordinance. Due to its composition no occupational risks are known. Occupational diseases or allergies were not determined by handling of this product. Note batch number and expiry date separately printed on the packaging. Expiry date: 36 months as of manufacturing date.

Certificate and information

For additional information on this product, please see → www.rheosol.com – For specific questions or information about expert reports, please contact our office staff: